



GAS PERMEABLE LENS CARE SYSTEM

**Patient Instruction Guide for
OCUVIQ[®] Conditioning Solution**

Consult an Eye Care Professional for suitability of wear

www.oculusgp.com

INSTRUCTIONS FOR USE

OCUVIQ® Conditioning Solution

100ml Conditioning and conservation solution – For rigid gas permeable contact lenses.

What is OCUVIQ® Conditioning Solution

OCUVIQ® Conditioning Solution is an aqueous solution for the conditioning and conservation of rigid and gas permeable contact lenses while they are not in use.

Mode of Use

1. Wash your hands thoroughly with mild soap.
2. To prepare your lenses for overnight storage, apply 2-3 drops of OCUVIQ® All-Lenses Cleaner to both sides of the lenses and rub gently with your thumb and forefinger for 15 seconds.
3. Rinse the lenses thoroughly with sterile saline solution and soak the lenses in OCUVIQ® Conditioning Solution for at least 6 hours or overnight before wearing.
4. Contact lenses can be inserted directly into the eyes after removing the lenses from the lens case.

Precautions and Warnings

1. Follow the mode of use.
2. Always use fresh OCUVIQ® Conditioning Solution to disinfect, soak and store contact lenses.
3. Keep bottle tightly closed when not in use and store at room temperature away from direct sunlight. To avoid contamination, do not touch dropper tip with your hands or against any surface.
4. Discard solution 90 days after opening.
5. Lenses can be stored for a maximum of 30 days in the closed lens case.
6. In case of discomfort or irritation, immediately remove the contact lens and consult an eyecare professional before resuming use.
7. Keep out of reach of children.
8. Not to be used in the eye.
9. Wash eyes if the solution comes in direct contact with eyes.

Contraindications

1. Do not use the product with soft contact lenses.
2. Do not use the product if you are allergic to any of its ingredients.
3. Do not use the product if the guarantee seal is broken or the packaging is damaged.

Adverse Reactions (Problems and what to do)

You may also experience the following reactions: stinging, burning or itching (irritation), reduced comfort compared to the first time you put the lenses on, a feeling of something in the eye (a foreign body, a scratch in a certain area), tearing eye wear, unusual discharge from the eye, red eyes, decreased visual acuity (reduced vision), blurred vision, glare or halos around objects, sensitivity to light (photophobia), or dry eyes.

If you experience any of these reactions: **Remove your lenses immediately and**

- If the discomfort or problem goes away, carefully examine the lenses up close.
- If the lens is damaged, do not put it back on your eye. Put the lens back in its case and contact your eye care professional.
- If the lens is dirty, has chips or a foreign body, or if the problem goes away and the lens is undamaged and has been thoroughly cleaned, rinsed and disinfected, put the lens back on.
- **If the problem persists, remove your lenses and see your eye care professional immediately.**

Adverse Effects Notification

You can notify us of medical incidents or adverse reactions at the web address: <https://www.oculusgp.com/>.








Throw Away

Dispose of the product in the usual way. Try to recycle cardboard cartoning and plastic packaging as indicated in your city.

Composition

Sodium Chloride, Boric Acid, Sodium Tetraborate, Hydroxypropyl Methylcellulose, Poloxamer, Edetate Disodium 0.1%, and Polyhexamethylene Biguanide 0.0004% in Purified Water.

Symbol Reference Guide on Packaging and Label

	Manufacturer		Quality system certification symbol
	Instructions for use or consult electronic instructions for use		Single sterile barrier system with protective packaging outside Sterilized using aseptic processing techniques
	Do not use if package is damaged and consult instructions for use		Medical Device
	Caution		

**Manufacturer:**

Eurolent Servicios Opticos y Oftalmológicos, SL
CL Pintores, 5
28923, Madrid, Spain

Product Owner/Legal representative

Oculus Visioncare (S) Pte. Ltd.
10 Kaki Bukit Avenue 1 #06-04
Singapore 417942
Tel: (65) 6742 8386